



## Carpo Flex-X Measuring Guide

**STEP 1.** Use a measuring tape or piece of string to measure around your dog's Ankle/Wrist area.

**See figure (A).**

**STEP 2.** Measure the distance from the back of your dog's paw-pad (or the floor) to about half way up your dog's leg.

**See figure (B).**



## **Sizes**

### **X-Small**

**3<sup>3</sup>/<sub>4</sub>" W x 3<sup>1</sup>/<sub>2</sub>" H**

Fits small dogs with **up to 3<sup>3</sup>/<sub>4</sub>"** wrist circumference

### **Small**

**4" W x 4<sup>1</sup>/<sub>2</sub>" H**

Fits small-medium dogs with **up to 4"** wrist circumference

### **Medium**

**5<sup>3</sup>/<sub>4</sub>" W x 6<sup>1</sup>/<sub>4</sub>" H**

Fits medium dogs with **up to 5<sup>3</sup>/<sub>4</sub>"** wrist circumference

### **Large**

**7<sup>3</sup>/<sub>4</sub>" W x 8" H**

Fits large dogs with **up to 7<sup>3</sup>/<sub>4</sub>"** wrist circumference

### **X-Large**

**8 1/2" W x 9" H**

Fits giant breed dogs with **up to 8 1/2"** wrist circumference

\*Due to sanitary concerns this item cannot be exchanged or returned once placed on an animal.